## Lab Dept: Chemistry

## Test Name: CASHEW COMPONENT Ana o 3

## **General Information**

| Lab Order Codes:                   | CCAO3   |
|------------------------------------|---|
| Synonyms:                          | N/A   |
| CPT Codes:                         | 86008 - Allergen specific IgE; quantitative or semiquantitative, recombinant or purified component, each  |
| Test Includes:                     | Cashew component Ana o3 reported in kU/L.   |
| Logistics                          |   |
| Test indications:                  | Detect allergen specific-IgE.   |
| Lab Testing Sections:              | Chemistry   |
| Phone Numbers:                     | MIN Lab: 612-813-6280   |
|                                    | STP Lab: 651-220-6550   |
| Test Availability:                 | Daily, 24 hours   |
| Turnaround Time:                   | 1 - 2 days, performed Monday - Friday   |
| Special Instructions:              | N/A   |
| Specimen                           |   |
| Specimen Type:                     | Blood   |
| Container:                         | SST Tube (Gold, marble or Red)  |
| Draw Volume:                       | 1.0 mL blood  |
| Processed Volume:                  | 0.3 mL serum  |
| Collection:<br>Special Processing: | Routine blood collection<br>Lab Staff: Centrifuge specimen, remove serum aliquot into plastic tube.<br>Store at refrigerated temperatures for up to 1 week. |
| Patient Preparation:               | None  |

## Interpretive

| Reference Range: | Cashew Components Ranges:<br>kUA/L: Interpretation:<br><0.1 *Absent<br>0.1 - 0.34 Equivocal/Low<br>0.35 - 0.69 Low<br>0.70 - 3.49 Medium<br>3.50 - 17.49 High<br>17.50 - 49.99 Very High<br>50.00 - 100.00 Very High<br>>100.00 Very High |
|------------------|---|
| Critical Values: | N/A   |
| Limitations:     | N/A   |
| Methodology:     | ImmunoCAP FEIA  |
| References:      | Thermofisher Scientific Allergen Encyclopedia (August 2024)   |